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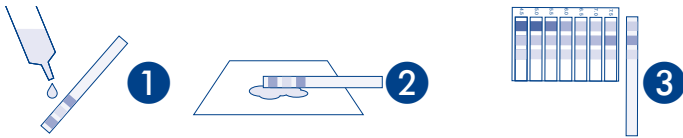
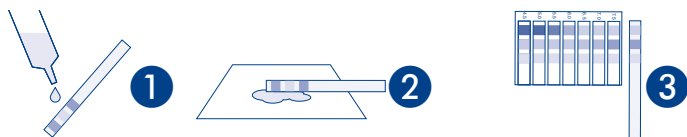
Tel.: 062 388 55 00 · sales-ch@mn-net.com

Rev 05/2023

Instructions for use: Test strips for the determination of pH in gastric juice in the range of pH 4.5–10.0

Assay procedure for measuring the pH value of gastric fluid:

1. Aspirate human gastric fluid sample and cover all test fields on the test strip.
2. Dab off excess liquid from the outer edge onto an absorbent paper.
3. Immediately compare with the color scale and read corresponding pH value.

Contents:

- 100 test strips
- 1 x instructions for use with comparison scale

Intended purpose:

pH-Fix 4.5–10.0 PT test strips for human gastric fluid are semi-quantitative indicator test strips for checking the pH of gastric fluid in the pH range of 4.5–10.0 with increments of 0.5 pH units. The evaluation is performed manually by visual comparison with the color scale. The results are used as an aid in the control of correct nasogastric feeding tube placement. The strips are for use by healthcare professionals. They are suitable for near-patient use outside of a laboratory. They are not suitable for self-testing.

Notes:

Do not touch the test fields.

Test strips are for single use only.

The use of the system is limited to samples that are not strongly colored.

Aspirate gastric fluid sample with a syringe from nasogastric feeding tube and test immediately. Sample volume must be sufficient to properly wet all test pads according to the assay procedure. If not sufficient sample volume is obtained or if the pH value is not in the range 1–5.5, then X-ray is required to confirm correct nasogastric feeding tube placement.¹⁾

Only for use outside of the body.

For quality control use commercially available pH buffer solution (e.g. Titrisol pH 7.00, article number: 109887, supplier: Supelco).

User with color vision deficiency must consult a person with normal color vision for evaluating the test result.

Some medication may have an influence on the pH-value of gastric fluid. For example, proton pump inhibitors (e.g. Omeprazole, Lansoprazole or Pantoprazole) raise the pH value in gastric fluid after some hours after intake.

Performance data:

A comparative study (N = 390) shows an average deviation of the test strip compared to the reference method (pH electrode) of 0.09 pH units. Batch to batch precision (repeatability) was determined to be 99.6 % in a study with 65 samples. Repeatability within series (N = 30) was 100 %. Reproducibility (N = 30) was 100 %.

Storage conditions:

Protect test strips from sunlight and moisture. Store in a cool and dry location (storage temperature 4–30 °C). If stored properly, the test strips can be stored until the printed expiry date.

Disposal:

Used test strips may contain potentially infectious material of human origin. Dispose all used test strips in accordance to the local laws and regulations.

Test principles:

The pH of a sample generates a color change in the pH indicator dye which is chemically bound to the paper of the test field.

Reactive substances:

Azo dye 15: 6 µg/cm² (test pad 1), azo dye 43: 9 µg/cm² (test pad 2), azo dye 18: 6 µg/cm² (test pad 3).

Reporting obligation:

We wish to point out that all serious incidents which occur in connection with the product must be reported to the manufacturer and the competent authority of the European member state or of the state in which the incident occurred. European vigilance contact points:

https://ec.europa.eu/health/md_sector/contact_en

Reason of revision:

Addition of performance data and vigilance contact points.

Literature:

- 1) NHS Improvement, Initial placement checks for nasogastric and orogastric tubes, 2016.
- 2) A. M. Bourgault, J. Heath, V. Hooper, M. L. Sole, E. G. NeSmith, *Critical Care Nurse*, 2015, 35.

Technical Service:

If you still have questions after reading the instructions or need technical assistance, please contact

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MACHEREY-NAGEL GmbH & Co. KG