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Intended purpose:

The URYXXON® control strip is used to verify the performance of the URYXXON® Relax, URYXXON® Relax (Connect) and URYXXON® 500 devices. The IVD demonstrates the measurement accuracy and linearity of the devices under test and is applied by laboratory personnel in a medical laboratory or by technicians in a laboratory setting.

Description:

The URYXXON® control strip is used to check the performance of the URYXXON® Relax, URYXXON® Relax (Connect) and URYXXON® 500 devices. It simulates a Medi-Test urine test strip and generates a high and medium reading in the control strip menu on the device for each wavelength evaluated. Thus, the URYXXON® control strip is used to check both the accuracy of the measurement and the linearity of the device under test. The results of the measurement are compared with the target values of the respective batch.

The strip must be manually placed on the instrument, feeding is not automatic. The reflectometric evaluation of the devices is performed quantitatively via the SEM value (reflectance). The URYXXON® control strip is intended for use by professional users.

Cautionary measure and warnings:

In vitro diagnostic medical device.

Observe the usual precautions when handling laboratory reagents. Please also observe the instructions for use for the URYXXON® reflectometers.

Disposal of all waste must be carried out in accordance with local guidelines.

Do not immerse the URYXXON® control strip in urine. The strip is intended for repeated use.

Protect the test fields from contamination and do not touch them with your fingers.

Components:

- 1 URYXXON® control strip
- 1 Instruction manual
- 1 Evaluation sheet

Additional materials needed:

- URYXXON® Relax (REF 93088), URYXXON® Relax (Connect) (REF 930880) or URYXXON® 500 (REF 930080)
- Protective gloves if necessary

Reagent handling:

The URYXXON® control strip is ready for use. If the control strip is kinked, damaged or soiled, it must not be used for the control measurement.

Storage and shelf life:

Protect the URYXXON® control strip from sunlight and moisture. Store the packaging in a cool, dry place (4 °C–30 °C, max. 80 % humidity, non-condensing). When not in use and at the end of the controls, immediately reseal the control strip to protect it from contamination.

When stored properly, the URYXXON® control strip is stable until the expiry date printed on it.

The control strips are disposed of with regular household waste.

Implementation on the URYXXON® Relax and the URYXXON® Relax (Connect):

NOTE: Clean the strip holder before using the URYXXON® control strip.

URYXXON® Relax as of software version 3.02 and the URYXXON® Relax (Connect) (all software versions):

1. Switch on the device using the I/O switch.
2. Tap the screen several times during the power-on process.
3. Enter service code 1111 via the numeric keypad and confirm with ✓.
4. Select „Control Strip“.
5. Place the URYXXON® control strip on the strip holder and perform the measurement.
6. Repeat the measurement three times (steps 4 and 5) and compare the values obtained with the target ranges.
7. Exit the service menu via ☒.
8. The device now starts in normal operation.

URYXXON® Relax from software version 1.62 to 2.23 (except 1.8D):

1. Select main menu ▼.
2. Select Settings in the main menu.
3. Select „Measurement“.
4. Select „Control strip“ and confirm with ✓.
5. Place the URYXXON® control strip on the strip holder and perform the measurement.
6. Repeat the measurement three times (steps 1 to 5) and compare the values obtained with the target ranges.

URYXXON® Relax up to software version 1.61 and 1.8D:

ATTENTION: The control strip is not supported by these versions. Please contact MACHEREY-NAGEL or your local distributor for further information.

Implementation on the URYXXON® 500:

NOTE: Before checking with the URYXXON® control strip, remove all strips from the collection container of the transport table and clean it.

URYXXON® 500 from software version 1.38:

1. Select the main menu.
2. In the main menu, select „Test mode“, then „C-Strip“.
3. Place the strip on the transport table, the strip is automatically drawn in and measured.
4. Enter the result in the evaluation sheet.
5. An SEM value is given for each LED used (orange, green, blue, infrared). In each case for a high and a medium value.
6. Repeat the measurement three times and compare the values obtained with the target ranges.
7. After the evaluation, exit the menu via ☒.

NOTE: Before taking a measurement with the URYXXON® Stick 10, remove the URYXXON® control strip from the collection container of the transport table.

URYXXON® 500 up to software version 1.38:

ATTENTION: The control strip is not supported by these versions. Please contact MACHEREY-NAGEL or your local distributor for further information.

Evaluation:

The measurement of the URYXXON® control strip on the reflectometers is carried out three times. For this purpose, the strip must be applied a total of three times. All determined reflectance values are compared with the tolerance range on the evaluation sheet. The values to be used are described in the „Performance“ chapter. If a measurement is outside the tolerance range specified on the evaluation sheet, the reflectometer concerned should no longer be used to analyze customer samples. In this case, contact MACHEREY-NAGEL or your local distributor.

Cleaning:

If the URYXXON® control strip becomes contaminated, it can be carefully cleaned with a cotton swab and a small amount of 70 % isopropanol or compressed air spray.

NOTE: Make sure that the surface is flat and clean.

NOTE: The URYXXON® control strip must not become too wet.

Note on mandatory reporting of incidents:

We would like to point out that all serious incidents that have occurred in connection with the product must be reported to the manufacturer and the competent authority of the European Member State or the state in which the incident occurred. European Vigilance Contact Points: https://ec.europa.eu/health/md_sector/contact_en

Notes:

The IFU in additional languages is available on the website www.mn-net.com.

Symbol explanation:

	Item number
	Batch Identification
	Expiration Date
	Temperature limit
	Please note the instructions for use!
	Manufacturer
	<i>In vitro</i> diagnostic medical device
	Indicates conformity with the applicable harmonized standards
	Control material

Revision:

12/2025

Reason for revision:

Addition of the “Notes” section.